

# SANIFECT

Universal agent for simultaneous cleaning and disinfection of surfaces

- **Effective against a broad range of microorganisms and viruses: being able to destroy Gram-positive and Gram-negative bacteria (incl. Mycobacterium Tuberculosis), deactivates viruses (HBV, HIV, Rota), also effective against pathogenic fungi genus Candida and dermatophyte genus Trichophyton.**
- **Simultaneous disinfection and cleaning effect.**
- **Aldehyde- and phosphate free.**
- **Suitable for pre-sterilization cleaning and disinfection of medical instruments.**



## Areas of application

Product is used for disinfection and cleaning of surfaces and equipment in medical institutions, veterinary centers, food processing areas, childcare facilities and everywhere else where disinfection is necessary. Suitable for the disinfecting cleaning of alcohol sensitive surfaces of medical equipment and surfaces.

**SANIFECT** – is a blue color liquid concentrate.

## Active ingredients

Alkyldimethylbenzylammonium Chloride - 4,5%, Alkyldimethylethylbenzylammonium Chloride - 4,5%.

## Packaging units

Plastic bottle 1 L; Can 5 L.

## Storage life

5 years in original package

## Labelling according Regulation (EC) No 1272/2008 [CLP]

Signal word - Danger

Hazard statement(s)

H302 Harmful if swallowed.

H314 Causes severe skin burns and eye damage.

Precautionary statement(s)

P270 Do not eat, drink or smoke when using this product.

P280 Wear protective gloves/protective clothing/eye protection/face protection.

P301+P330+P331 IF SWALLOWED: Rinse mouth. Do NOT induce vomiting.

P303+P361+P353 IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water/shower

P305 + P351 + P338 IF IN EYES: Rinse cautiously with water for several minutes.

Remove contact lenses, if present and easy to do. Continue rinsing.



## Tested in accordance with European standards

EN 1275:2006, EN 1276:2009, EN 1650:2008+A1:2013, EN 13624: 2013, EN 13697:2015, EN 13727:2012+A2:2015, EN 14348:2005, EN 14561:2006, EN 14562:2006, EN 14563:2009, EN 14476:2013+A1:2015.

## Efficiency

Microorganism	Concentration	Contact time
Bacteria (incl. MRSA, Pseudomonas aeruginosa) and yeast (C.albicans)	0,4% 0,8%	60 min 10 min
Dermatophyte - Trichophyton	0,8%	10 min
Mycobacterium - M. Tuberculosis	2,3%	60 min
Enveloped viruses (incl. HIV, HBV, HCV, Vaccina, Herpes, Influenza)	0,8%	10 min
Enteroviruses (incl. HAV, Adeno и Polyo)	3,8%	240 min

## Sanitary washing and disinfective cleaning of surfaces

Concentration of preparate	Contact time
0,8%*	10 min

\* 60 ml of concentrate + 7,5 L of water



European Union  
European Social Fund



Investing in your future



LETRINA S.A.  
ISO 9001



LETRINA S.A.  
ISO 14001



EN ISO 13485



EN ISO/IEC 17025  
L236

Baltiachemi OÜ  
Maagi 5  
74114 Maardu, Estonia  
Tel. +372 621 4694  
info@baltiachemi.ee  
www.baltiachemi.ee