

RAPIQUAT

Ready-to-use agent for rapid disinfection of surfaces, medical devices and instruments

- **Effective against a broad range of microorganisms and viruses:** being able to destroy Gram-positive and Gram-negative bacteria (incl. *Mycobacterium Tuberculosis*), deactivates viruses (HBV, HCV, HIV etc), as well as yeast and fungi
- **Convenient preparative forms**
- **Rapid disinfection of surfaces and medical devices, including alcohol-sensitive surfaces**
- **Entirely harmless to people and the environment**



Application areas

The agent is intended:

- For quick cleaning and disinfection of medical equipment - electrocardiographs, ultrasonic sensors, incubators for newborns, dental equipment, including patient chairs, various types of acrylic and plastic medical devices, including equipment wires.
- For quick disinfection of surfaces and instruments in cosmetology and dermatology clinics, hairdressers, nail salons, tattooists, solariums, as well as in other areas of activity associated with damage or risk of damage to the skin of visitors and staff.

RAPIQUAT - is a clear, slightly colored liquid with a specific odor.

Active ingredients

Didecyl dimethyl ammonium chloride - 0,45%, N-(3-Aminopropyl) dodecylpropane diamine - 0,25%, Polyhexamethylene biguanide hydrochloride - 0,04%.

Packaging

Plastic bottle with trigger – 500 ml, nonwoven impregnated wipes in plastic tube 100 pcs.

Storage life

2 years in original package

Tested in accordance with European standards

EN 1275:2005, EN 1276:2009, EN 13624:2013, EN 13697:2015, EN 13727:2012+A2:2015, EN 14348:2005, EN 14561:2006, EN 14562:2006, EN 14563:2009, EN16615:2015, DVV / RKI :2008.

Efficiency

Microorganism	Contact time
Bacteria and yeast (incl. MRSA, <i>C.albicans</i>)	30 sec
Mycobacterium <i>M.Terrae</i> , <i>M.Tuberculosis</i>	15 min
Parenteral viruses (incl. HBV, HCV, HIV)	30 sec

Modes of processing

Rapid cleaning and disinfection of surfaces of equipment and tools

Contact time - 30 seconds

Cleaning and disinfection for increased requirements for tuberculosis prevention

Contact time - 15 minutes



Baltiachemi OÜ
Maagi 6A
74114 Maardu, Estonia
T. +372 621 4694
info@baltiachemi.ee
www.baltiachemi.ee